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Ingredients: PMA-Zeolite®

Intended use:

For absorbing exudates and forming a protective layer on superficial skin injuries.

PANACEO CARE-ZEOLITE WOUND POWDER absorbs exudates and forms an even, flexible protective layer on intact skin and superficial wounds. As a rule, this prevents the formation of a brittle crust, providing the wound with optimal protection against mechanical influences. This creates a stable wound environment that supports natural healing.

Application:

Apply PANACEO CARE-ZEOLITE WOUND POWDER thinly and evenly on the affected area. Do not apply to open, deep, or infected wounds. Cover with a bandage if necessary. Repeat application once or several times a day or as directed by a medical personal and continue until the wound has completely healed. Suitable for all age groups!

Safety instructions:

Not a substitute for medical treatment. Do not use on deep or gaping wounds. Do not inhale. Do not use in the nose or ears. Avoid contact with the eyes. In case of serious incidents, please contact the manufacturer and your local authority. Keep out of reach of children under 6 years of age. Always close the can after use. Dispose of the product in accordance with local regulations with household waste.

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PANACEO International GmbH